

**PSP Single Use Medical Latex Gloves**

Code: PSP 50-190

Size: XS, S, M, L, XL

Imported by: President Safety B.V.,
Postbus 100, 3220 AC Hellevoetsluis / NL
www.psp-safety.com



1. Before use, read the instructions carefully and make sure the products are with integrated package and within validity date.
2. These non-sterile gloves can only be used in occasions where sterilization is not required.
3. Please take of rings, ornaments and cut nails neatly before wearing gloves.
4. Wear gloves in appropriate size and adjust in order to make the gloves fit appropriately for comfort, dexterity and sensitivity.

Attention

Used gloves shall be disposed as medical waste in order to prevent environmental pollution caused by bacteria.

Caution!

These gloves are made from natural rubber latex which can cause allergic reactions, including anaphylactic responses.

Storage

1. This product should be stored in a dry, cool place away from direct sunlight and cannot get in touch with oil and acid.
2. This product should be stored at a temperature of 5-35 °C, relative humidity of 50-80% is recommended. Keep the products away from ozone and ultraviolet rays which are harmful to latex cuticle.

Validity Date

Please see the date of manufacture and the date of expiry printed on the box.

Manufactured in accordance with
BS EN455-1, 2, 3 & 4.

MADE IN CHINA

